

TRICVALVE® TRANSCATHETER BICAVAL VALVES CT PROTOCOL

CAVI – VISUALIZATION OF VENA CAVA INFERIOR AND SUPERIOR

IMAGING PREPARATION

- ✓ Patient's arms up
- ✓ Angio CT from V.subclavia to renal veins

PROTOCOL ANGIO CT THORAX

- ✓ Extinct automatic breath hold command
- ✓ Delay Zero
- ✓ Flow 5ml/70ml contrast (app. 14 sec.)
- ✓ Start breathhold
- ✓ Start injection of contrast after 2 sec. (6-8 sec.)

